


Refer to the IFU for the detailed instructions of all the procedures described in the Quick Guide.


Odne™ Fill

STEP 1

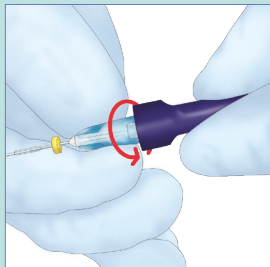


Connect the OdneFill Tip to the syringe.

STEP 2

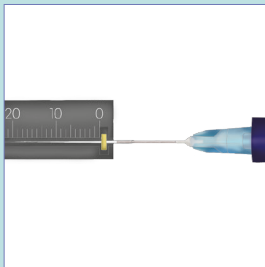

Measure the working length.


STEP 3

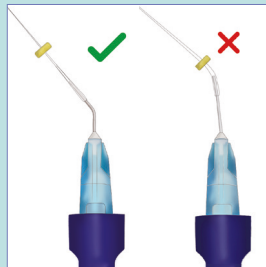

Bend the tip from the metal section to the desired position.

Odne™ Cure


STEP 4

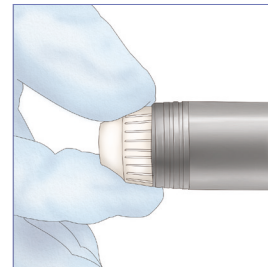

Remove the protective screw cap.


continue in the next steps →

STEP 5

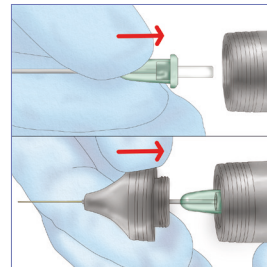


Insert the OdneCure Tip into the handpiece. Screw back the cap.

STEP 6

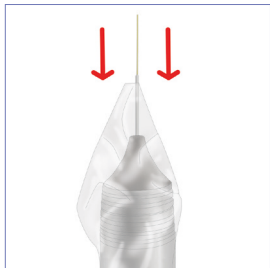


Cover the OdneCure handpiece with a protective sleeve.

STEP 7

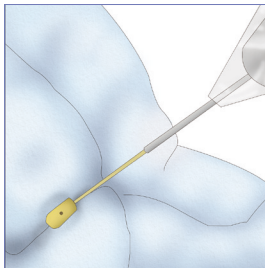

Insert the rubber stopper into the OdneCure Tip.

STEP 8

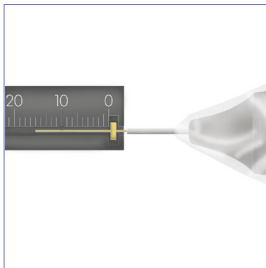


Position the rubber stopper 3-5 mm short of the working length.

STEP 9



Carefully bend the tip from the metal section to the desired position.


STEP 10

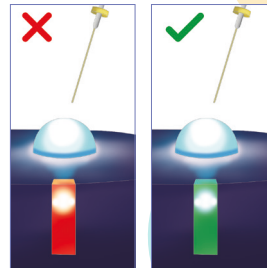

Verify the optical power emission.


Refer to the IFU for the detailed instructions of all the procedures described in the Quick Guide.

STEP 1

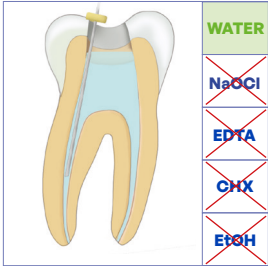

Rinse the root canals thoroughly with water.


STEP 2

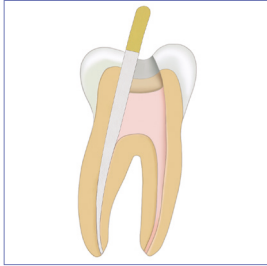

Dry the canals with a paper point.


Odne™ Fill

STEP 3

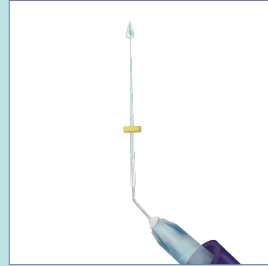


Discard the first drops of OdneFill to remove the air from the tip.

STEP 4

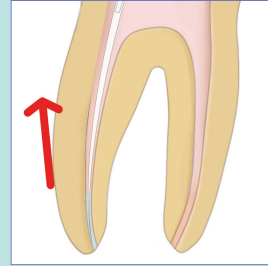

Insert the tip to 1 mm above the apex. Apply OdneFill while withdrawing the tip.


STEP 5

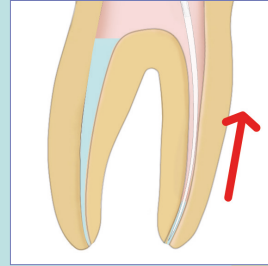

Repeat for all root canals.


continue in the next steps →

STEP 6

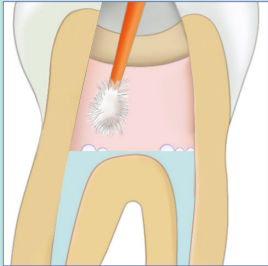

Using a microbrush, or a micro suction, remove the bubbles or excess material from the pulp chamber.


STEP 7

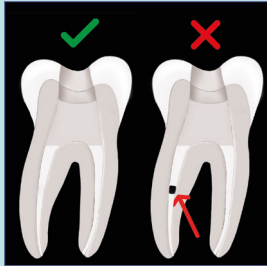

Perform an X-ray. If a bubble is present, reapply OdneFill so the bubble raises to the top. Repeat Steps 6 and 7.


Odne™ Cure

STEP 8

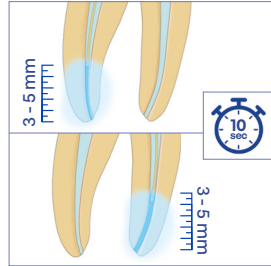

Insert OdneCure Tip 3-5 mm from working length. Turn the lamp ON for 10s (1 click). **Repeat for all root canals.**


STEP 9

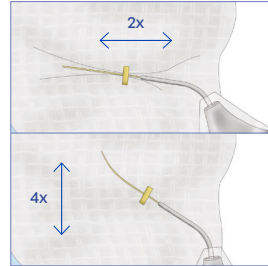

After retrieving the tip from the last canal, fold an alcohol-soaked wipe (e.g., CaviWipe*) and wipe it thoroughly, both on the sides and on the top.

* Registered trademark of Metrex Research, LLC

STEP 10

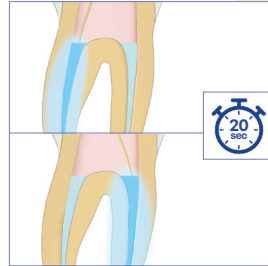

Place the tip near the canal opening. Turn the lamp ON for 20s (2 clicks). **Repeat for all root canals.** Perform a final x-ray.