



Refer to the IFU for the detailed instructions of all the procedures described in the Quick Guide.


PREPARATION FOR USE

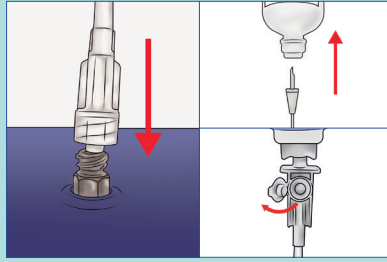


STEP 1: Connect the saline bag to the device using the inlet line. Open the side vent on the inlet line's spike.

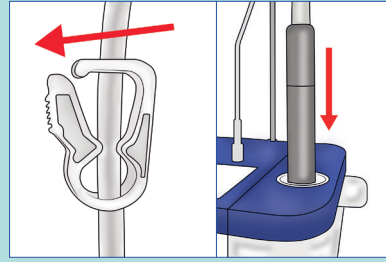


STEP 2: Insert the handpiece in the disinfection chamber and press the PRIMING button.

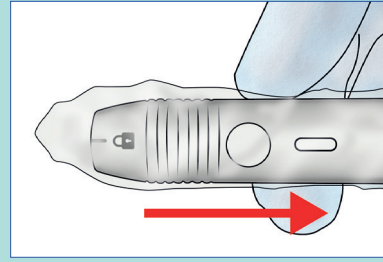

STEP 3: Cover the handpiece with a new protective barrier sleeve.

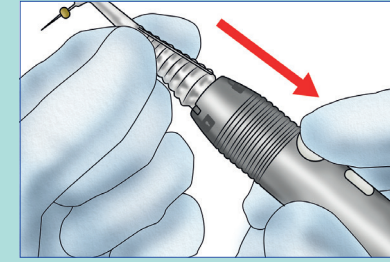


STEP 4: Attach an OdneClean Tip to the handpiece. Align the tip wings with the locked icon.

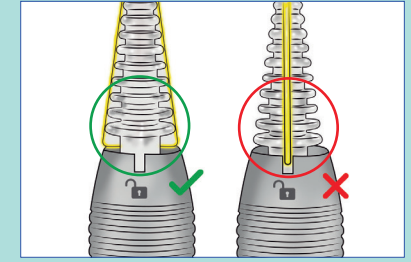


CAUTION: Do not push the wings of the tip all the way into the lateral slots of the handpiece!

OPERATION

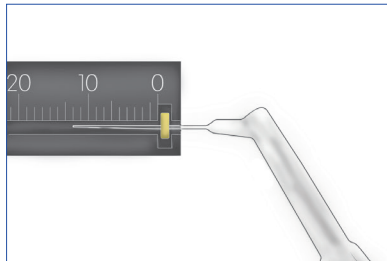


STEP 1: Position the rubber stopper on the OdneClean Tip 3 mm short of the root canal length.

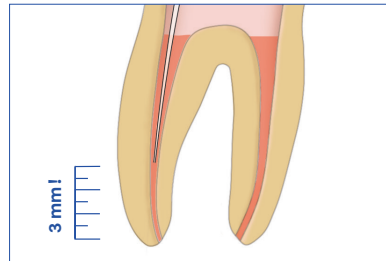


STEP 2: Insert the OdneClean Tip inside the root canal.



STEP 3: Press the handpiece button once to start a 30 seconds cycle. (See IFU for operating modes).

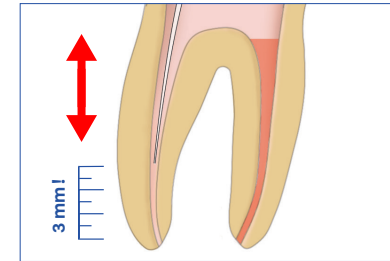


STEP 4: Gently move the tip in a pecking motion inside the canal. Ensure that the tip remains 3 mm away from the apex at all times.

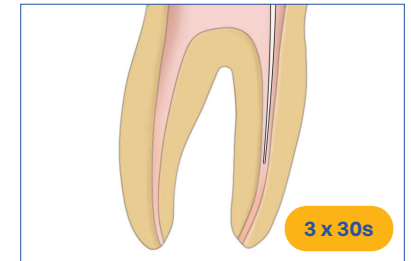


STEP 5: Perform 3 x 30 seconds cycles in each root canal that requires treatment following steps 2 to 4.


AFTER USE

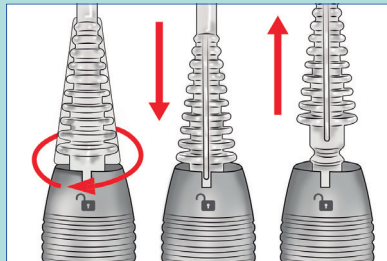


STEP 1: Rotate, press the tip into the lateral slots of the handpiece, and then pull to remove it.

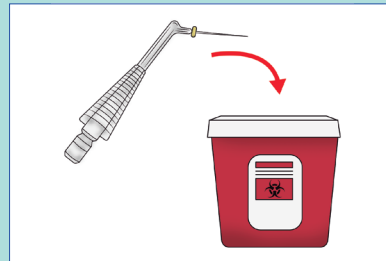


STEP 2: Dispose of the tip.

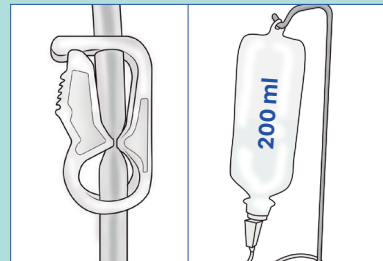

STEP 3: Check if the saline bag contains at least 200 ml of solution. Otherwise replace it.

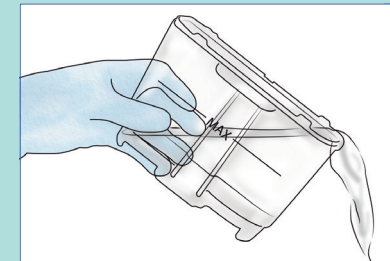


STEP 4: EMPTY the waste container and RINSE it with water.

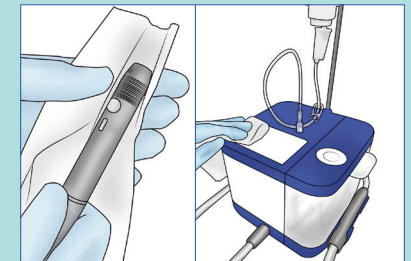

STEP 5: Clean and disinfect all the surfaces according to Chapters 5.2 and 5.3 of the IFU.


DISINFECTION (DAILY)

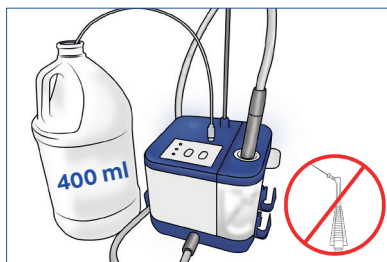


STEP 1: Place the handpiece without the tip into the disinfection chamber. Connect a bottle of disinfectant with at least 400 ml (14 FL OZ).




STEP 2: Press the DISINFECTION BUTTON for at least 3 seconds. Wait 15 minutes.

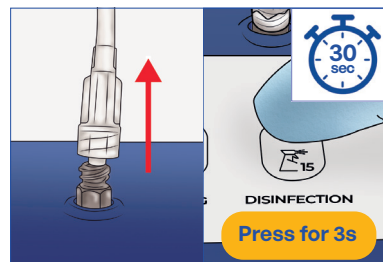


STEP 3: Disconnect the disinfectant hose and press the DISINFECTION button for at least 3 seconds.

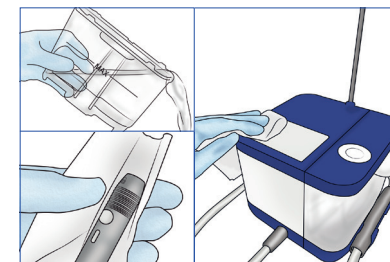


STEP 4: Clean and disinfect all the surfaces according to Chapters 5.2 and 5.3 of the IFU.

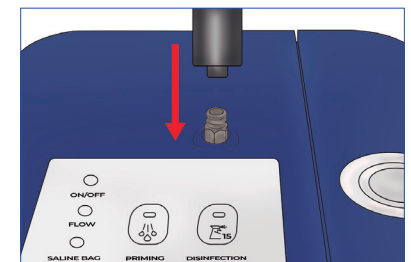

STEP 5: Place the disinfected cap on the inlet port.

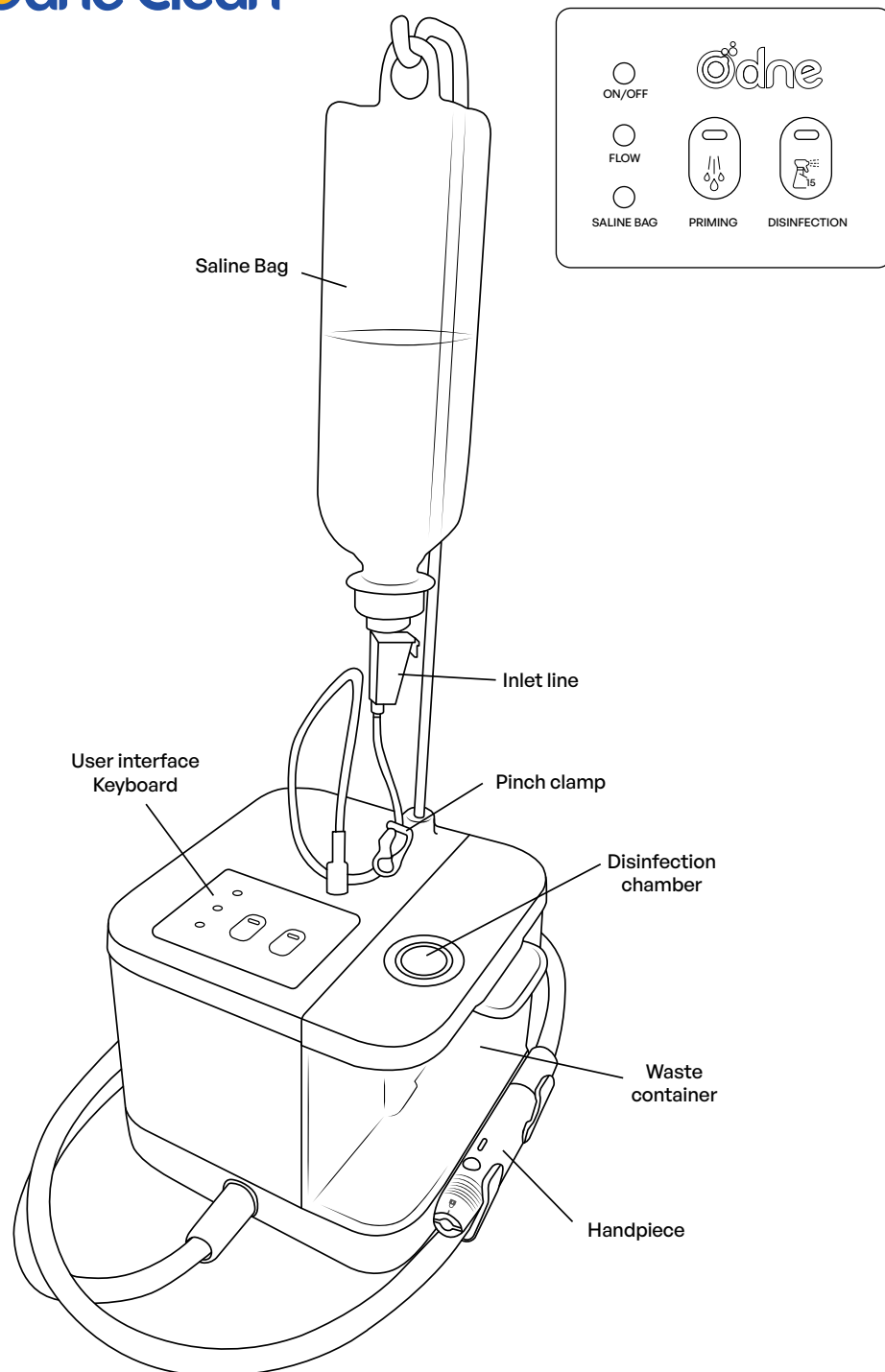

Icon	Led Color	Audio	Meaning	Solution
ON/OFF	Green	-	The device is powered on.	-
	Orange	-	Maintenance error.	Contact the manufacturer.
Flow	Steady Blue	-	The “30 seconds mode” treatment mode is active.	-
	Flashing Blue	-	The “Continuous mode” treatment mode is active.	-
	Red	3 fast beeps	During treatment: blocked or broken tip. During priming or disinfection: Attached tip.	Stop treatment. Retrieve and replace the tip. Remove the tip.
Saline bag	Steady Orange	3 slow beeps	Pinch clamp closed or empty saline bag.	Open the clamp or replace the saline bag. If the error persists, contact the manufacturer.
Priming	Steady Orange	-	The priming mode is active.	-
	Flashing Orange	-	Priming required before the device can be used.	Perform a priming sequence.
	Flashing Orange (after Red Flow LED)	3 fast beeps	The tip is attached.	Remove the tip. Re-start the priming sequence.
Disinfection	Steady Orange	-	The disinfection mode is active.	-
	Flashing Orange	-	Waiting to flush the disinfectant out of the system.	Disconnect the disinfectant hose and bottle, and press the button again.
	Flashing Orange (after Red Flow LED)	3 fast beeps	The tip is attached.	Remove the tip. Re-start the disinfection sequence.
Saline bag + Priming	Flashing Orange	3 slow beeps	Pinch clamp closed or empty saline bag.	Open the clamp or replace the saline bag. If the error persists, contact the manufacturer.
Saline bag + Disinfection	Flashing Orange	4 alternating tones (repeating for 15 minutes)	Pinch clamp closed, empty disinfectant bottle or the disinfection hose is not submerged in the disinfectant.	Open the clamp, replace the disinfectant bottle or submerge the hose. If the error persists, contact the manufacturer.

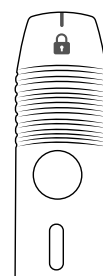

Led Color	Audio	Meaning	Solution
Steady Blue	-	The “30 seconds mode” treatment mode is active.	-
Flashing Blue	-	The “Continuous mode” treatment mode is active.	-
Red	3 fast beeps	Blocked or broken tip.	Stop treatment. Retrieve and replace the tip.